## PORTUGUÊS

- Material: Alumínio/Stainless
- Limite de Peso: 45kg
- Peso: 105g

	Torque	Loctite
Parafuso do Pé	20Nm	243

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**EN - Caution:** Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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For secure fastening use the loctite recommended and apply rated torque twice.

Note: A bonding agent may be required to secure the adapter to the foot, depending on the SACH foot selected. If in doubt contact the SACH foot manufacturer.

It is essential that only the supplied Foot Bolt is used to secure the SACH Foot Adapter in position, providing a secure thread engagement. If necessary the Foot Bolt should be cut to length.

- 2 year limited warranty

Aluminum tested to ISO10328, load level A60, rated to 100lbs/45kg for normal use.

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© Copyright Össur 2010 Rev 4